Instrucciones
Manuale d'uso
Bruksanvisning

Gebrauchsanweisung
Operating manual
Mode d'emploi

Legende

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Legend

1. facial cushion
2. Head rest – Upholstery frame
3. Chest cushion
4. Arm rest
5. Sitting cushion
6. Knee cushion
7. Strap – adjustment of arm rest
8. Adjustment of working height
9. Adjustment of inclination angle, head rest
10. Adjustment of inclination angle

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1. Adjustment of sitting height

- Push in press button (8)
- Adjust height
- Let the press button snap into place (check !)

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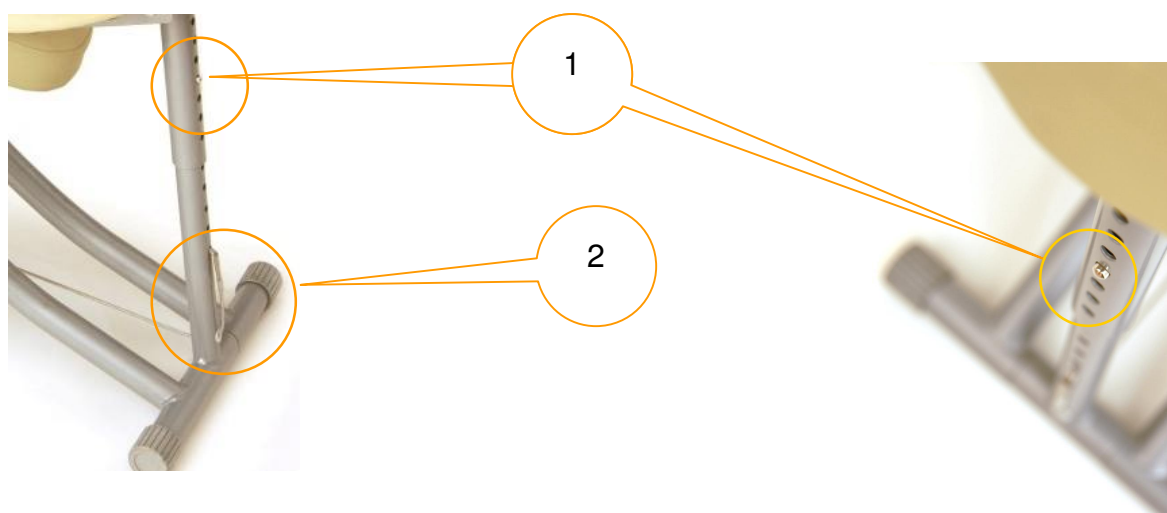
2. Adjustment of inclination angle

- Set the massage chair aside
- Plug end of cord (10) in another opening
 - up = acute angle, short distance
 - down = obtuse angle, long distance
- Lift chair, make sure to position it securely (check !)

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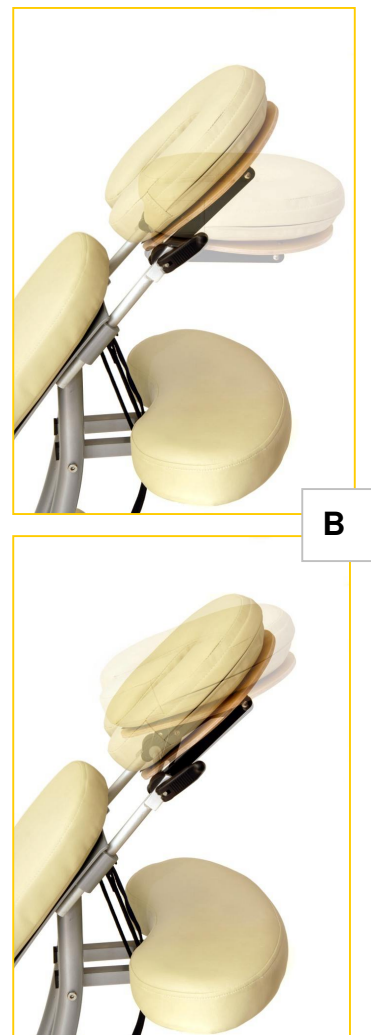
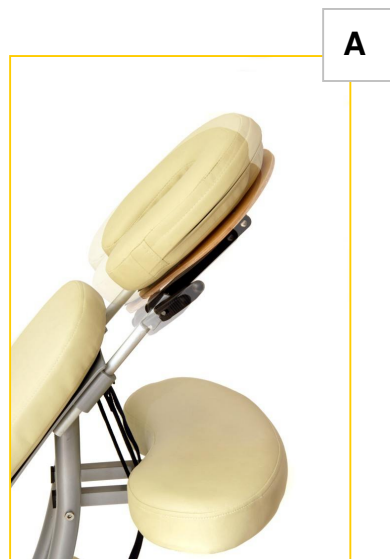
Adjustment of inclination angle, head rest (9)

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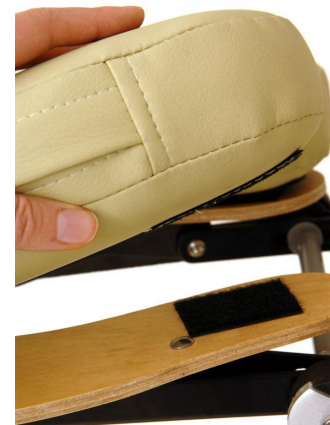
1. Warning notice

1. Check your **SISSEL® Portal pro** for damages prior to every use.
2. All screws and knobs must be tightened securely. Plastic-, metal pieces and cushions must be free of tears.
3. Keep your fingers our of moving parts / joints
4. Place your **SISSEL® Portal pro** onto a stable surface or support. Ensure a secure, anti-slip position.

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2. Cleaning

Clean your **SISSEL® Portal pro** with warm soapsuds or a mild, alcohol-free cleaning agent. Use a moist cloth and dry off with a clean rag afterwards.

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SISSEL - Productservice AG